ORBIS

MTA - DirectCaps

EN	DE	DA	SV	NO
FI	PL	LT	ET	LV

EN	DE	DA	SV	NO
FI	PL	LT	ET	LV

CE	MD
0482	

EN	Instructions for Use
-----------	-----------------------------

ORBIS MTA - DirectCaps | Endodontic Repair Cement

Product description
ORBIS MTA is a biocompatible endodontic repair cement in DirectCaps. ORBIS MTA powder is consisting of very fine hydrophilic particles of several mineral oxides. After contact with ORBIS MTA liquid it forms a gel that hardens to an impermeable barrier. ORBIS MTA is delivered in DirectCaps. DirectCaps are easily activated and the content of the capsule is easily ejected out with the DirectGun. Capsule mixing (mixing time 30 seconds) is achieved by a high frequency mixer with about 4.300 oscillations/min.

Indications/Intended use

- Repair of root perforations (during root canal therapy)
- Root end filling (retrograde)
- Pulp capping
- Apexification / Root end filling (orthograde)

Performance features
The performance features of the product meet the requirements of the intended use.

Contraindications
None known

Patient target group
Persons who are treated during a dental procedure.

Intended users
This medical device should only be used by a professionally trained dental practitioner.

Side effects
None known

1. Application
1.1. Activation and Mixing (see instruction for DirectCaps)
Activate and mix the DirectCap according to the information in the DirectCap instruction.
Mixing time for the DirectCaps is 30 seconds.

Attention:
Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting and lag times may impair or prevent application of the material. The material must be extruded within 10 seconds after the end of mix.
To prevent dehydration during setting, apply ORBIS MTA intraoral immediately after mixing.
Working time (at 23°C) of ORBIS MTA is approx. 2:00 minutes from start of mix.

2. Application
2.1. Repair of root perforations
Place rubber dam and clean the root canal system using intracanal instruments and NaOCl solution. Dry the root canal with paper points and isolate the perforation. Fill the root canal apical of the perforation completely with a suitable root canal filling material. Mix ORBIS MTA DirectCaps as described under point 1 and extrude it on a glass plate. Apply ORBIS MTA with suitable instruments into the perforation site and condense it. Check the position of ORBIS MTA in the root canal by an X-ray. If an adequate barrier has not been created, rinse ORBIS MTA out of the canal and repeat the procedure.
Remove excess moisture with a cotton pellet or a paper point.

Place a cotton pellet in the access to the root canal and apply a temporary filling material. Alternatively seal the root canal with a suitable root canal filling material and seal the cavity with a tight filling. Both options can be done at the earliest 5 minutes after placement of the ORBIS MTA.

ORBIS MTA repair material remains as a permanent part of the root canal filling.

2.2. Root End Filling (retrograde)
Create a surgical access to the root-end and resect the root. Prepare an apical cavity to a depth of 3-5 mm. Isolate the area and dry the root end cavity with paper points. Achieve hemostasis with suitable methods. Mix ORBIS MTA as described under point 1 and extrude it on a glass plate. Apply ORBIS MTA with suitable instruments and condense it using a small plugger. Remove excess cement and clean the surface of the root with a moist piece of gauze. Confirm placement of the ORBIS MTA repair material with an X-ray. ORBIS MTA remains as a permanent part of the root canal filling.

2.3. Pulp Capping
Place rubber dam and prepare the cavity. Rinse the cavity and exposed pulpal areas with a suitable disinfectant. Mix ORBIS MTA as described under point 1 and extrude it on a glass plate. With a suitable instrument apply a small amount of ORBIS MTA over the exposed pulp and remove excess moisture with a cotton pellet. At the earliest 5 minutes after application of ORBIS MTA place a small amount of a flowable light cure liner and light cure. Etch the remaining cavity walls according to the total-etch-technique with etching gel (e.g. ORBIS Etching gel) and apply a suitable bonding agent (e.g. ORBI-BOND TE) according to the corresponding instructions. Place a light cure composite (e.g. ORBI-HIFI SupremeX) according to the instructions and light cure. Check pulp vitality and status regularly.

2.4. Apexification / Root End Filling (orthograde)
Place rubber dam and clean the root canal system using intracanal instruments and irrigate with NaOCl. Dry the root canal with paper points. For disinfection place calcium hydroxide paste in the root canal and seal the access opening with a temporary filling material.

Alternatively seal the root canal with a suitable root canal filling material and seal the cavity with a tight filling.
Additional Notes/Warnings

- In the first hour after application handle the placed MTA cement carefully.
- Store ORBIS MTA in the sealed packaging in a dry place prior to use.
- ORBIS MTA can cause discoloration.
- Keep away from children!

Composition
MTA, bismuth oxide
Storage
Store at 10-25 °C (50-77 °F) in the original seal-pack and protected from moisture. Do not use after expiry date.
DirectCaps are for single use only.

Disposal
Disposal of the product according to local authority regulations.
Reporting obligation
Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note
The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – https://ec.europa.eu/tools/eudamed).

Warranty
First Scientific Dental GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. Use is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and First Scientific Dental GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability
Except where prohibited by law, First Scientific Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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