## Retraction cords

|           |                        |   |
|-----------|------------------------|---|
| <b>EN</b> | Instructions for use   | 2 |
| <b>DE</b> | Gebrauchsanweisung     | 2 |
| <b>DA</b> | Brugsanvisning         | 3 |
| <b>NO</b> | Bruksanvisning         | 3 |
| <b>FI</b> | Käyttöohje             | 4 |
| <b>PL</b> | Instrukcja użytkowania | 4 |
| <b>LT</b> | Naudojimo instrukcija  | 5 |
| <b>ET</b> | Kasutusjuhend          | 5 |
| <b>LV</b> | Lietošanas instrukcija | 6 |
| <b>NL</b> | Gebruiksaanwijzing     | 6 |
| <b>CS</b> | Návod k použití        | 7 |
| <b>FR</b> | Mode d'emploi          | 7 |

Medical device | Medizinprodukt | Medicinsk udstyr | Medicinteknisk produkt  
Medisinske utstyret | Lääkinnällinen laite | Wyrób medyczny | Medicinos priemonė  
Medtsiiniseade | Medicinskā ierīce | Medisch hulpmiddel | Zdravotnický prostředek  
Dispositif médical | Dispositivo medico | Tæki fyrir tannlækningar

Nur für dentale Zwecke | For dental use only | Kun til dental anvendelse | Kun til dentale formal | Endast för dentala ändamål | Vain hammaslääketieteelliseen käyttöön  
Tylko do celów dentystycznych | Skirta naudoti tik odontologijoje | Kasutamiseks ainult hambaravis | Tikai lietošanai zobārstniecībā | Alleen voor tandheelkundig gebruik  
Pouze ke stomatologickému použití | Réserve à l'usage dentaire

**ORBIS Dental**  
**Handelsgesellschaft mbH**  
Schuckertstraße 21 | DE-48153 Münster  
[www.orbis-dental.com](http://www.orbis-dental.com)

**Müller-Omicron GmbH & Co.KG**  
Schlosserstr. 1 | DE-51789 Lindlar  
Germany

## EN Instructions for use

### Knitted retraction cord; non-impregnated

Read this instructions for use carefully before use! ORBIS retraction cord is intended exclusively for dental use by qualified personnel. ORBIS retraction cord is used to displace the gingival tissue on the prepared tooth to improve the result of the impression.

#### Composition:

100 % cotton

#### Application

1. Check the condition of the gum on the tooth to be applied, such as depth or thickness.
2. Use after checking the expiration date of the products.
3. Select the appropriate thickness of the cord for the area to be used. Open the lid, pull out the cord to the required length for the tooth circumference and cut the cord with the integrated cutter by closing the lid.
4. Insert the cord into the gingival sulcus by using a cord packer or another suitable instrument. (e.g. ORBIS cord layer)
5. Shortly before taking the impression, remove the retraction cord.

#### Intended Purpose:

1. Intended use: Used to retract the gingiva before taking an impression and remove residual saliva, and tissue fluids in the gingival sulcus.
2. Indication: Apply to the gingival tissue that requires dental impressions.
3. Contraindication: Do not insert the cord deeply for patients with periodontal disease. The use of the inflamed area shall be considered by weighing the benefits against the risk.
4. Patient population: All age group.
5. Intended user: Dentist.
6. Side effects/Adverse effects: There are no side effects or adverse effects reported.
7. Potential risk if re-used: Re-use of cut retraction cord may cause inflammation.
8. Can be used with retraction cord packer and hemostatic agent, and it shall be used under the control of the dentist according to the clinical indication.
9. Single-use device.

#### Preventive measures:

1. Do not re-use the cut retraction cord, single-use only.
2. Check the product packaging abnormality before use.
3. It is prohibited to use it for any purpose other than the designated purpose of use.
4. Do not use the products that have expired.
5. Dental professional use only.
6. There may be permanent gingival recession if applied for more than 20 minutes, and when taking the impression, the cord must be removed.
7. Do not insert the cord deeply for patients with periodontal disease.
8. Remove the cord slowly to prevent force application on the gingival tissue.
9. The use of ORBIS retraction cords on inflamed areas should only be considered under risk-benefit assessment.

#### Storage:

1. Close the cap immediately after use.
2. Store at room temperature in a dry place, away from direct sunlight.

#### Disposal:

Dispose of contents/container according to the related regulations of the country.

#### Shelf Life:

5 years from the date of manufacture.

**In case of any serious incident that has occurred in relation to the device, please report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.**

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