Plaque Disclosing Pellets, Blue

- EN
- DE
- DA
- SV
- NO
- FI
- PL
- LT
- ET
- LV
- NL
- CS
- FR
- IT

EN

ORBIS Plaque Disclosing Pellets, Blue

PRODUCT INFORMATION

General info:
Single-use product supplied non-sterile. Intended for temporary use only.

Re-use & End of Life:
The product life cycle ends after use, as the product is not for re-use.

Special Equipment & Accessories:
To be picked up with a pair of tweezers.

Material:
High absorbent blue pellets (cellulose and viscous fibres) made of Water, Glycerine, Erythrosine, Ethanol, Peppermint oil, Ethyl Paraben Sodium Salt.

Blue pellets contain the organic colouring agent Brilliant Blue FCF. Gluten-free.

Acceptable daily intake (ADI) according to FAO/WHO is well below the limit values for all the components in the pellets; 0,1 mg Erythrosine, 12,5 mg Brilliant Blue and 10 mg Ethyl Paraben Sodium Salt / day and kg body weight, i.e., much higher than the exposure with the pellets.

INTENDED USE & INSTRUCTIONS

Intended use:
Soaked pellets used to reveal dental plaque. Single-use.
For professional use only. No additional training is needed.

- Instructions:**
1. Apply Vaseline on the patient's lips (to avoid discoloring of the lips).
 2. Pick up the pellet with a pair of tweezers.
 3. Brush the soaked pellet over the teeth surfaces.
 4. Let the patient rinse the mouth with water.
 5. Show the patient with a mirror where cleaning has been effective / ineffective.
 6. Let the patient clean off the colour from the teeth.
 7. Teach the patient how to clean the teeth.

LIMITATIONS & RESTRICTIONS

Notifications:
Note that sterilization is not possible.

Cautions:
Be careful to spill on clothes and hands, the stains can be difficult to wash off.

OTHER

Transport Conditions:
Transport accordingly to secure clean & safe condition.

Storage Conditions:
Store accordingly in dark, dry and clean area. In refrigerator or room temperature.

Shelf Life:
Minimum 3 years in room temperature (20°C).

Disposal instructions:
Disposal in accordance with local regulations.

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SVENSKA DENTORAMA AB Sollentunav. 130 191 44 Sollentuna/Stockholm, Sweden
Plandent Oy Asentajankatu 6 FI-00880 Helsinki Finland www.plandent.com