Instructions for use

1. Advantages:

- Super flexibility
- Excellent cutting efficiency
- Reduce the risk of contamination
- Simplicity, one file system
- Longer fatigue life
- Shorten the preparation time

2. Data:

Size	Model	Taper
Small (yellow)	20#	.07
Primary (red)	25#	.07
Medium (green)	35#	.06
Large (white)	45#	.05

3. Motion:

Reciprocating motion. Left cutting, 150 degrees left and 30 degrees back.

4. Sequence:

- Irrigate the canal and coated files with EDTA.
- Use hand K file #10 to explore the root canal, estimate the working length.
- X-Ray evaluation to confirm the width, length of the root canal.
- Established straight-line coronal and access.
- Explore the canal by hand use K file #10, to confirm the glide path.
- Expand this glide path to at least #15 by path files.
- Always use the file (25/07 red) as your first file for shaping of the canal by taking it to length with small in-and-out motions advancing the file apically 1–3 mm per stroke.
- Continue with the file (25/07, red), in 2–3 passes, to pre-enlarge the coronal two thirds of the canal.
- Carry the file (25/07, red) to the full working length in one or more passes. Upon reaching length, remove the file to avoid over-enlarging the tip. Inspect the apical flutes; if they are loaded with debris, then the shape is finished.
- If the file (25/07, red) meet pressure, then use the file (020/07, yellow) in one or more passes to working length and then use the file (25/07, red) to working length to optimize the shape.
- If the files (25/07, red) goes down to working length without resistance, no debris out, finish with either the file (35/06, green) or file (45/06, white).

5. Precautions:

- 5.1 Orbis Golden One files are sterile files.
- 5.2 Use the same motor, hand piece and settings as WaveOne Gold Files to shape canals in a reverse-reciprocating motion and settings exactly like WaveOne Gold.
- 5.3 Use an in-and-out motion advancing the file 1–3 mm with each stroke.
- 5.4 Irrigate the root canal according to the appropriate irrigation protocol.
- 5.5 Orbis Golden file is only for single use.

For dental use only!

orbis-dental.de/goldenone

STERILE 

Plandent Oy
Asentajankatu 6
00880 Helsinki
www.plandent.com

Shenzhen Superline Technology Co., Ltd.
Room 1206, Building 4,
Chongwen Garden, Taoyuan Street,
Nanshan District, Shenzhen 518055
Guangdong P.R. China.
www.supline.com

Wellkang Ltd.
The Black Church, St. Mary's Place,
Dublin 7, D07 P4AX, Ireland

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