
ORBI-Sept IMD

Abformdesinfektion

Impression disinfection

- en: ORBI-Sept IMD impression disinfection
 da: ORBI-Sept IMD desinfektion af aftryk
 sv: ORBI-Sept IMD avtrycksdesinfektion
 no: ORBI-Sept IMD avtrykkdesinfeksjon
 fi: ORBI-Sept IMD jäljennösten desinfektioaine
 pl: ORBI-Sept IMD środek do dezynfekcji mas wyciskowych
 lt: ORBI-Sept IMD atspaudų dezinfekavimo priemonė
 et: ORBI-Sept IMD jälgimise desinfitseerimine
 lv: ORBI-Sept IMD iespieddezinfekcija
 nl: ORBI-Sept IMD desinfectie van afdrukken
 cs: ORBI-Sept IMD Sdezinfekce otisků
 fr: ORBI-Sept IMD désinfectant pour empreintes
 it: ORBI-Sept IMD disinfezione dell'impressione

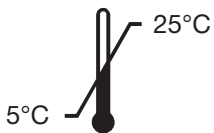UFI: 4KX2-KA80-400N-07XT

ORBIS

ORBIS DENTAL Handelsgesellschaft mbH
 Schuckertstr. 21 | D-48153 Münster | Germany
www.orbis-dental.com

REF 262558

MD

CE 0124

2,5 L e

2026-03

PRISMAN GmbH
 Otto-Hahn-Ring 6-18
 D-64653 Lorsch
 Phone: +49 (0)6251-866 980-0

de

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en

Intended use:

Ready-to-use, aldehyde-free impression disinfectant for dental impression materials.

Instructions for use: Rinse the impression in question first with water, then insert it into the ready-to-use disinfectant solution and remove it after the recommended contact time. The impression must be completely covered with the solution. Immerse alginates and hydrocolloids only briefly, then allow to dry. Carefully rinse the impression afterwards with a soft jet of water. Do not mix the solution with cleaners or other disinfectants. The working solution must be changed at least every working day if visibly soiled and in accordance with RKI recommendations. It can be disposed of via the wastewater. For professional users only.

Contact times:

Bacteria (incl. MRSA) according to VAH (Association of Applied Hygiene), EN 13727, EN 14561 (dirty conditions):	Conc. 5 min.
Candida albicans according to VAH, EN 13624, EN 14562 (dirty conditions):	Conc. 5 min.
„Limited virucidal“ (enveloped viruses such as HBV, HIV, HCV) according to EN 14476 and EN 17111 (clean conditions):	Conc. 5 min.
TB (M. terrae in suspension test, clean conditions):	Conc. 15 min.

Safety instruction:

H 412 Harmful to aquatic life with long lasting effects. P273 Avoid release to the environment. P501 Dispose of contents / container in accordance with local / regional / national / international regulations.

All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

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