ORBIS))) Tray Cleaner Konzentrat

- de** Konzentrat zur schonenden Reinigung von dentalen Abformlöffeln und Instrumenten.
- en** Concentrate for gentle cleaning of dental impression trays and instruments.
- da** Konzentrat til skånsom rengøring af tandaftryksbakker og tandlægeinstrumenter.
- sv** Konzentrat för skonsam rengöring av dentala avformningsskedar och instrument.
- no** Konsentrat for skånsom rengjøring av tannavtrykksskjeer og instrumenter.
- fi** Tiiviste jäljennöslusikoiden ja instrumenttien hellävaraiseen puhdistukseen.
- pl** Konzentrat do delikatnego czyszczenia stomatologicznych łyżek wyciskowych i instrumentów.
- lt** Koncentratas medžiagas tausojančiam odontologinių atspaudų šaukštų ir instrumentų valymui.
- et** Hambajäljendite ja instrumentide õrnatoimeliseks puhastamiseks ette nähtud kontsentraat.
- lv** Koncentrāts zobu nospiedumu paliktņu un instrumentu saudzīgai tīrīšanai.
- nl** Concentraat voor de behoedzame reiniging van tandheelkundige afdruklepels en instrumenten.
- cs** Koncentrát pro šetrné čištění dentálních otiskovacích lžic a nástrojů.
- fr** Concentré pour le nettoyage en douceur des porte-empreintes et instruments dentaires.
- it** Concentrato per la pulizia delicata di portaimpronte dentali e strumenti.

Gefahr / Danger
 Farlig / Fara / Fare /
 Vaara / Niebezpieczeństwo
 Pavojinga / Ettevaatust
 Bīstami / Gevaar / Nebezpečí
 Danger / Pericolo

UFI: 1VF5-DUUV-5102-AKQT

REF 193082

1 L e

2026-03

ORBIS)))

ORBIS DENTAL Handelsgesellschaft mbH
 Schuckertstr. 21 | D-48153 Münster | Germany
www.orbis-dental.com

PRISMAN GmbH
 Otto-Hahn-Ring 6-18
 D-64653 Lorsch
 Phone: +49 (0)6251-866 980-0

de

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en

Type of application:

Concentrate for the gentle removal of zinc oxide-eugenol pastes, carboxylate and phosphate cements, and alginate residues from metal and plastic surfaces. The concentrate is also suitable for removing thermoplastic impression materials and waxes and can also be used in an ultrasonic bath at room temperature.

Instructions for use:

The Tray Cleaner is used as a 5-10% solution in water. The contaminated parts are best placed in the ready-to-use dilution and left to soak for at least 45 minutes, depending on the degree of contamination. The special cleaner is suitable for non-rusting objects. Chrome-nickel and aluminum parts should not be cleaned together. Risk of corrosion! When pre-cleaning invasive medical devices, they must be hygienically reprocessed in accordance with the manufacturer's instructions before reuse. For professional users only!

Safety instructions: H 318 Causes serious eye damage. H 226 Flammable liquid and vapor. P 210 Keep away from heat/sparks/open flames/hot surfaces. — No smoking. P 233 Keep container tightly closed. P 280 Wear protective gloves/protective clothing/eye protection/face protection. P 305+P351+P338 F IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P 337+P 313 If eye irritation persists: Get medical advice/attention. P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

648/2004/EC: Between 5% and 15% anionic surfactants, fragrances.

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