ORBIS

CompuLe Dispenser Gun

ISO 13485

EN Instructions for Use

Name

Product Name: Applicator gun for dental materials in compules

Structure, performance and applicable scope of product

The product is a tool for injecting dental material during dental treatment. The plastic part is made of PA, and the metal connecting parts are stainless steel, which has no toxic and side effects on the human body. It delivers resin material into the patient's tooth cavity when repairing teeth.

Use Method:

1. Make sure the product is clean before use.
2. Install the injection tip with dental resin in the slot on the front of the instrument, and then repeatedly lock the front and rear handles to push the resin material forward to the target position.

Caution:

1. It is forbidden to use when the product is damaged or deformed.
2. It is forbidden to use when the package is damaged.
3. Please do not use this product outside the original purpose of this product.

Storage and Maintain Method:

1. This product should be stored in a clean, dry, ventilated place with room temperature, relative humidity not greater than 80%, and no corrosive gas.
2. Avoid direct sunlight.
3. Prevent external extrusion and deformation during transportation and storage.

Contraindication:

No contraindication is identified by now.

Reprocessing instructions:

The dispenser gun is autoclavable and washable in a thermal disinfectant. It can be used repeatedly

and must be heat sterilised at 134 °C before use. Please disassemble the product as described below.

Lift the clip. Disassemble first, then sterilise.

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Plandent Oy Asentajankatu 6 FI-00880 Helsinki Finland www.plandent.com