ORBIS)))

ORBI-Temp NE

EN **DE** **DA** **SV** **NO** **FI**

PL **LT** **ET** **LV** **NL** **FR**

CE⁰⁴⁸²

EN **Instruction for use**

[MD] EU Medical device In accordance with DIN EN ISO 3107

Product description: ORBI-Temp NE, a eugenol-free cement with calcium hydroxide, is easy to process and does not influence the setting of resinbased filling and luting materials.

ORBI-Temp NE contains calcium hydroxide to support tooth vitality.

Indications:

ORBI-Temp NE is used as a temporary cement for the attachment and for sealing of small one-surface cavities.

Contraindications:

ORBI-Temp NE contains modified colophony. ORBI-Temp NE should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

Patient target group:

ORBI-Temp NE is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product’s performance features satisfy the requirements of the intended use and the relevant product standards.

User:

ORBI-Temp NE should only be used by a professionally trained dental practitioner.

Use:

Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Place and lock the mixing tip with a 90° turn clockwise. The material is automatically mixed in the mixing tip.

All syringes come with a detachable piston. The piston can simply be pulled from the syringe and re-placed without causing any material leakage. The plugs in the syringe chambers remain in place preventing cross-contamination of the materials.

Application:

Apply a thin layer of ORBI-Temp NE inside the temporary crown or bridge and affix it to the prepared tooth.

ORBI-Temp NE remains workable for about 1 min. ORBI-Temp NE sets in the mouth within 4 - 6 min.

Place the temporary into the mouth. Remove any excess material after 3 - 5 min.

Warnings, precautionary measures:

– In case of opened pulp or inferior dentine thickness, an adequate pulp protection is recommended.

– The sensitivity of cores can be reduced by use of desensiti-
zing preparations prior to temporary luting. Remove these
preparations completely before definite luting (do not use
preparations containing eugenol!). Extensive superficial and
multisurfaced cavities as well as temporary longterm sealing
work should be treated with a harder cement.

– Our information and/or advice do not relieve you of the
obligation of checking that the products supplied by us are
suitable for their intended purpose.

Supplementary indications:

Residues of ORBI-Temp NE can be removed from crowns easily after they have been sprayed for a short time with a cooling spray.

Storage:

Store at 4°C-23°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient’s, user’s or other person’s health condition and a serious risk to public health that arise or could have arisen in association with the use of ORBI-Temp NE must be reported to the manufacturer and the responsible authority.

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MP Medical Product GmbH