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|             |                                           |
|-------------|-------------------------------------------|
| <b>EN</b>   | <b>Instructions for use</b>               |
| <b>[MD]</b> | <b>EU Medical device</b>                  |
|             | <b>In accordance with DIN EN ISO 4049</b> |

**Product description:**
**ORBIS BulkFill** is a light-curing, highly radiopaque hybrid composite designed specifically for time-saving cost-effective use in the posterior region. The combination of a new multi-hybrid filler technology with an innovative initiator system formed the basis for a filler material exhibiting minimal polymerisation shrinkage and excellent depth of cure. **ORBIS BulkFill** can even be cured reliably in increments of 4 mm and with very short polymerisation times. **ORBIS BulkFill** undergoes reduced shrinkage stress to ensure restorations with integral margins. The physical strength and high abrasion-resistance are high enough for occlusal load-bearing fillings. **ORBIS BulkFill** contains 86 % by weight inorganic filler (= 70,1 % by volume) in a methacrylate matrix. **ORBIS BulkFill** is cured with halogen or LED light with a blue spectrum and is available in practical syringes or Caps for direct application.

**Shade:**
Universal (U)

**Indications:**

- Class I and II posterior fillings
- Core build-up

**Contraindications:**
**ORBIS BulkFill** contains methacrylate and BHT. **ORBIS BulkFill** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

**Patient target group:**
**ORBIS BulkFill** is suitable for use in all patients without any age or gender restrictions.

**Performance features:**
The product's performance features satisfy the requirements of the intended use and the relevant product standards.

**Use:**
**ORBIS BulkFill** should only be used by a professionally trained dental practitioner.

**Use:**
**Preparation:**
Clean the teeth being treated. If applicable, mark occlusal contact points. Bring material to room temperature before use.

**Cavity preparation:**
It is essential to prepare the tooth minimally-invasively (adhesive fillings), preserving healthy tooth structure. Clean and dry the cavity afterwards.

**Creating a dry working field:**
Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

**Matrix placement:**
Take appropriate measures for areas where the applied **ORBIS BulkFill** can flow out. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and placement of the matrix.

**Pulp protection:**
A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

**Bonding materials:**
**ORBIS BulkFill** applied with a dentine enamel bond, using the adhesive technique. Any light-curing bonding materials can be used. Observe the relevant instructions for use during preparation (etching technique) and processing.

**Application:**
**Syringe:** Apply the **ORBIS BulkFill** and adapt it with a suitable instrument.
**Caps:** Insert **ORBIS BulkFill Caps** into the opening of the dispenser (follow instructions for use). Turn the cap into desired direction and remove the protective top. Apply **ORBIS BulkFill** directly into the cavity by slowly and evenly pressing the levers of the dispenser together.

**Light-curing:**
**ORBIS BulkFill** is specially formulated for applying in layers of up to 4 mm. Higher fillings must be built up in layers and each layer cured separately with a polymerisation lamp:

10 s when using lamps rated at 800 mW/cm² or higher.

20 s when using lamps rated at 500 to 800 mW/cm².

The outlet of the light guide should be positioned as close as possible to the restoration. Ensure that every section of the filling is fully exposed to the light. An additional polymerisation of the accessible case from the oral or vestibular side is recommended. Remove metal matrices before hand. If the light outlet is more than 5 mm from the surface, it must be assumed that the depth of cure will be reduced and that only that area actually illuminated by the light cone will be cured. Inadequate curing may lead to discolouration and pulpal complications.

**Finishing:**
After removing the matrices, the filling can be trimmed and polished immediately while being cooled (e.g. fine or extra-fine grit diamonds, polisher). The tooth should be fluoridated as a final step.

**Warnings, precautionary measures:**

- Each **ORBIS BulkFill Caps** unit should be used for one patient only and should be applied exerting uniform but not excessive pressure.
- Phenolic substances, especially preparations containing eugenol or thymol, inhibit the curing of **ORBIS BulkFill**. Therefore, do not use zinc oxide eugenol cements or other eugenol-containing substances in combination with **ORBIS BulkFill**.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

**Constituents (in descending order according to content):**
Barium aluminium borosilikat glass, BisGMA, UDMA, silicon dioxide, barium sulphate, TEGDMA, initiators, stabilisers, pigments

**Storage instructions and application method:**
Storage at 4°C-23°C. May be refrigerated. Seal the syringes immediately after removing the material in order to avoid exposure to light and subsequent polymerisation. Do not use after the expiry date.

**Disposal:**
Dispose of the product in accordance with local regulations.

**Reporting obligation:**
Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **ORBIS BulkFill** must be reported to the manufacturer and the responsible authority.

**Note:**
The Summary of Safety and Clinical Performance of **ORBIS BulkFill** can be found in the European database on medical devices (EUDAMED – https://ec.europa.eu/tools/eudamed).

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