**ORBIS** 

# Golden Taper

## EN Instructions for use

Before using the Orbis Golden Taper, please see the instruction manual as indicated below.

Size	Length	Apical Diameter	Torque	Speed	Cross Section
SX	19mm	0.19mm/0.04V	3.0N/cm	300rpm	
S1	21,25,31mm	0.18mm/0.02V	1.5N/cm	300rpm	
S2	21,25,31mm	0.20mm/0.04V	1.5N/cm	300rpm	
F1	21,25,31mm	0.20mm/0.07V	1.5N/cm	300rpm	
F2	21,25,31mm	0.25mm/0.08V	2.5N/cm	300rpm	
F3	21,25,31mm	0.30mm/0.09V	2.5N/cm	300rpm	
F4	21,25,31mm	0.40mm/0.08V	2.5N/cm	300rpm	
F5	21,25,31mm	0.50mm/0.05V	2.5N/cm	300rpm	

### Warnings

- Only skilled dentists are allowed to use.
- Be sure to sterilize this product for each use.
- Do not use this product except for the dental service and treatment.

Use it in accordance with the intended use.

### Contraindications and prohibitions

- Do not use this product for a patient who indicates sensitization and allergic reaction.

### Structures and principals

- These instruments are to be used only in a clinical or hospital environment, by qualified users. Application field: shaping and cleaning of the root canal system. The instruments are delivered sterile.

### 2. Main materials

- Working Cutting part: nickel-titanium alloy with heat treated.
- Shank: Stainless (Nickel, Chrome, Gold included).

### Purpose of use and effect-efficacy

This is the dental instrument with columnar or tapering working part and it possesses circular form in cross cut section and flat end.

### Specifications

Under the provisions of ISO 3630 performance test, product durability for twisting and bending must exceed specified value of our standard.

### Method of use

According to its intended use, choose the most appropriate Orbis Golden Taper file and use it for enlarging and debriding root canal.

### Precautions for use

- Multiple use disinfection and re-sterilization cycles may lead to increased risk of file separation.
- These instruments should not be immersed in a sodium hypochlorite solution.
- Instrument decontamination: strictly follow decontamination instructions from the manufacturer.
- Irrigate abundantly and frequently.
- Establish a glide path using hand files with a minimum size of ISO 15.

- Use in a constant rotation at a speed of 250-350 rpm with light apical pressure.
- Clean flutes frequently and check for signs of distortion or wear.
- For optimal usage, torque control devices are recommended.
- Use the Shaping Files (SX, S1 and S2) with a brushing action on the withdrawal stroke in order to create straight line radicular access.
- Use the Finishing Files (F1-F5) with no brushing action.
- Use the appropriate finishing files to passively follow the canal to the working length, and then withdraw immediately.

### Storages and duration of use

- Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- Do not damage or make a pinhole to packaging materials.
- This product is subject to be improved without previous notice. Enterance first-in first-out method for stock management.
- Do not store the instrument under a germicidal lamp to avoid deterioration.

### Maintenances and inspections

- Sterilize this product by autoclave under established method and term for each use.

### Method of sterilization:

Put this product in a sterilization pack (or foil) and place it on a sterilization tray for autoclave sterilization with reference to the following terms.

### Terms for sterilization:

- Sterilize with steam at 134°C (273°F) / 2,1 bars / 18 min. for endodontic instruments and filling.
- Do not use high pressure steam sterilizer which heats more than 200 degrees Celsius including drying process.
- When reusing the instrument, wash away foreign substances completely and sterilize.
- Regarding use of medical cleaning agent, follow the instruction manual by its manufacturer strictly.
- Dispose the product if damaged or decreased in performance.

### Packagings

- Minimum packaging unit: 6 pcs. /package.
- Assortment: 1 pcs. from each size/package (SX, S1, S2, F1-F5).

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ECO REP SUNGO Europe B.V.