ORBIS Orbis MTA - DirectCaps

GB Orbis MTA - DirectCaps

Instructions for Use
Endodontic Repair Cement

Orbis MTA is an endodontic repair cement in DirectCaps. Orbis MTA powder consists of very fine hydrophilic particles of several mineral oxides. After contact with Orbis MTA liquid it forms a gel that hardens to an impermeable barrier.

Orbis MTA is delivered in DirectCaps. DirectCaps are easily activated and the content of the capsule is easy to eject with the DirectGun. Capsule mixing (mixing time 30 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/min such as Capmix.

--	--

- Indications**
 - Repair of root perforations during root canal therapy
 - Root end filling (retrograde)
 - Pulp capping
 - Root end filling (orthograde)

- Contraindications**
 - Not known

- Side effects**
 - Not known

- Activation and Mixing (see Instruction for DirectCaps)**
 - Activate and mix the DirectCap according to DirectCap instructions.
 - Mixing time for the DirectCaps is 30 seconds.**

ATTENTION:

Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting and lag times may impair or prevent application of the material. The material must be extruded within 10 seconds after mixing. To prevent dehydration during setting, apply Orbis MTA intraoral immediately after mixing.

Working time of Orbis MTA is approx. 2:00 minutes (at 23°C).

5. Application

5.1. Repair of root perforations

Place rubber dam and clean the root canal system using intra-canal instruments and irrigate with NaOCl. Dry the root canal with paper points and isolate the perforation. Fill the apical canal space up to the perforation completely with a suitable root canal filling material. Mix Orbis MTA as described under point 4 and extrude it on a glass plate. Apply Orbis MTA with suitable instruments into the perforation site and condense it. Check the position of Orbis MTA in the root canal with an X-ray. If an adequate barrier has not been created, rinse Orbis MTA out of the canal and repeat the procedure. Remove excess moisture with a damp cotton pellet or a paper point. Place a damp cotton pellet in the access of the root canal and apply a temporary filling material. Alternatively seal the access preparation with a suitable root canal filling material and seal the cavity with a tight filling. Neither option can be done earlier than 5 minutes after the placement of the Orbis MTA. Orbis MTA repair material remains as a permanent part of the root canal filling.

5.2. Root End Filling (retrograde)

Create an access to the root-end and resect the root with a surgical bur. Use an ultrasonic tip to prepare a class I root-end cavity preparation to a depth of 3-5 mm. Isolate the area and dry the root end cavity with paper points. Achieve hemostasis with suitable methods. Mix Orbis MTA as described under point 4 and extrude it on a glass plate. Apply Orbis MTA with suitable instruments and condense it using a small plugger. Remove excess cement and clean the surface of the root with a moist piece of gauze. Confirm placement of the Orbis MTA repair material with an X-ray. The Orbis MTA repair material remains as a permanent part of the root canal filling.

5.3. Pulp Capping

Place rubber dam and prepare the cavity outline. If caries is present, remove it. Rinse cavity and exposed pulpal areas with a suitable disinfectant. Mix Orbis MTA as described under point 4 and extrude it on a glass plate. With a suitable instrument, apply a small amount of Orbis MTA over the exposed pulp and remove excess moisture with a dry cotton pellet. Wait 5 minutes after the application of Orbis MTA and place a small amount of a flowable light cure liner and light cure. Etlch the remaining cavity walls according to the total-etch-technique with ETCHING GEL and apply a suitable bonding agent according to the agent's instructions. Place a light cure composite according to the instructions and light cure. Pulp vitality and status should be checked by X-ray at regular intervals.

5.4. Root End Filling (orthograde)

Place rubber dam and clean the root canal system using intra-canal instruments and irrigate with NaOCl. Dry the root canal with paper points. For disinfection place calcium hydroxide paste in the root canal for one week. Seal the access opening with a temporary filling material. Mix Orbis MTA as described under point 4 and extrude it on a glass plate. With a suitable instrument, apply a small amount of Orbis MTA into the apical region and condense it. Create a 3 – 5 mm barrier of Orbis MTA. Check the position of Orbis MTA with an X-ray. If an adequate barrier has not been created, rinse Orbis MTA out of the canal and repeat the procedure. Remove excess moisture with a damp cotton pellet or a paper point. Place a damp cotton pellet in the access of the root canal and apply a temporary filling material. Alternatively seal the access preparation with a suitable root canal filling material and seal the cavity with a tight filling. Neither option can be done earlier than 5 minutes after the placement of the Orbis MTA. Orbis MTA repair material remains as a permanent part of the root canal filling.

Additional remarks

In the first hour after application handle the placed MTA cement carefully. Store Orbis MTA in the sealed packaging in a dry place prior to use. Intraoral application of Orbis MTA must be done immediately after mixing to prevent dehydration during setting. Orbis MTA can cause discoloration. Use Orbis MTA only in the root canal and/or the pulp chamber. In order to obtain a creamy consistency, 1 or 2 drops of sterile water can be added to the mixed Orbis MTA cement.

Orbis MTA

Storage
Store Orbis MTA in a dry place at 10 – 25 °C. Do not store below 10°C! Do not use after expiry date. DirectCaps are for single use only.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH’s sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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GB Instruction for activating and mixing DirectCaps

DirectCap before activation.
For activation of the DirectCap press the plunger on a hard and plane surface to the end into the DirectCap.
Insert the DirectCap into the DirectGun and **click once** to standardize.
Note: The plunger must be at the same level as the bottom of the capsule. Insert the DirectCap into a mixer (or an amalgamator), close lid and mix immediately for 30 seconds (about 4300 oscillations / min).
Remove the pin from the nozzle. If not, capsule can burst.
Insert the DirectCap into the DirectGun. Pull the lever 2 times (2 clicks) to prime the DirectCap. Extrude the mixed material on a glass plate and apply directly. Unlock the gun (push button A) and remove the DirectCap.

Only with the DirectGun the optimal amount of mixed material is guaranteed.

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