ORBI-HIFIL EASY

Lichthärtendes Universal-Composit

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ORBI-HIFIL EASY

Light Cure Universal Composite

ORBI-HIFIL EASY is a visible light cure universal composite for anterior and posterior restorations. It is based on methacrylate-resin and inorganic filler particles of 0.05-1.5 µm. The total filler load is 81 %, the total filler volume is 65 %. The composite material meets the requirements of: DIN EN ISO 4049.

Indications of ORBI-HIFIL EASY

Suitable for all cavities

Side Effects

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Preparation of Tooth and Color Matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching.

Ascertain the tooth shade while teeth are still moist and select the appropriate ORBI-HIFIL EASY shade.

ORBI-HIFIL EASY is shaded according to VITA®-Shades.

Cavity Preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction.

For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Apply a common adhesive for enamel and dentine bonding according to the corresponding manufacturer instructions. The application of the self etch adhesive ORBI-Bond SE ONE is recommended. Follow the instructions for ORBI-Bond SE ONE.

Application and Curing of ORBI-HIFIL EASY

Apply ORBI-HIFIL EASY in the selected shade. ORBI-HIFIL EASY is easy to model. Place it by using a suitable instrument. Matrix strips may be used.

In case of deep and extended cavities best results are obtained with a build-up of a base liner using a light cure resin modified cement or a light cure flowable composite (e.g. ORBI-Flow), followed by the application of ORBI-HIFIL EASY.

In case of small cavities ORBI-HIFIL EASY can be applied directly.

For an optimum result apply ORBI-HIFIL EASY in **layers of max. 2 mm**. Light cure each increment separately.

By using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm², cure each increment separately.

Lighter shades (e.g. A2, A3) **20 seconds**

Darker shades (e.g. A3.5) **30 seconds**

Finishing

Contour with fine diamonds, stones or burs. Polish to high gloss with discs or rubber points. Interproximal finishing is accomplished by fine grit finishing strips.

Storage

Do not store above 25°C (77°F). Protect from direct sunlight. Do not use after expiry date.

Close syringe immediately after use to avoid exposure to light.

Additional Notes

- Do not use any resin to adjust viscosity of composite restorative material.
- Contact of resin pastes with skin should be avoided, especially by anyone having known resin allergies.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.

Warranty

First Scientific Dental GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and First Scientific Dental GmbH's sole obligation shall be repair or replacement of the First Scientific Dental GmbH product.

Limitation of Liability

Except where prohibited by law, First Scientific Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!
For dental use only!