EN DE SV DA NO FI

ORBIS GI-Cem Luting Resin

INSTRUCTIONS FOR USE
MD EU Medical device
 In accordance with DIN EN ISO 9917-2, class 1

0482

Product description:
 ORBIS GI-Cem Luting Resin is a radiopaque, chemically curing, resin-modified glass ionomer luting cement in easy-to-use application capsules. The characteristics of the material are easy handling, low solubility in the mouth and high self-adhesion. Each capsule contains a minimum of 0.14 ml of usable cement.

Indications:

- porcelain-fused-to-metal crowns, bridges on hard dental tissue and on cores rebuilt with amalgam, composites or glass ionomer materials
- metal inlays, onlays, crowns and bridges
- root posts
- all-ceramic crowns manufactured from sintered ceramic, zirconium dioxide-only or alumina-only ceramic cores
- orthodontic appliances

Method of Use:

Thoroughly clean and dry enamel, dentine and cavity surfaces. Clean cores before luting (e.g. pumice, water), thoroughly rinse and dry with a gentle stream of air or cotton pellets (do not overdry).

Before application, the capsule has to be activated, mixed and opened.

Activating the application capsule and mixing the material:

Only extract the application capsule from the tubular bag immediately before use. During activation, the liquid from the plunger must be pressed into the inside of the capsule as fully as possible. To achieve this, the end of application capsule where the plunger of liquid protrudes is slowly pressed down onto a hard surface (e.g. a table) as far as the plunger will move (see fig. 1). The capsule must only be activated directly before use and loaded immediately into a high-frequency mixer with approx. 4.000 oscillations per minute. Mix the activated application capsule for 10 s, remove it from the mixer without delay and insert it in the holder of an application device (see corresponding instructions for use).

Application:

Open the outlet by lifting the cannula until you feel a stop (see fig. 2), and apply the material within 10 s of the end of mixing by compressing the lever on the applicator slowly and evenly. The cannula can be aligned as required by turning the whole application capsule inside the holder. Working time and consistency may be influenced by the respective type of capsule mixer and mixing time.

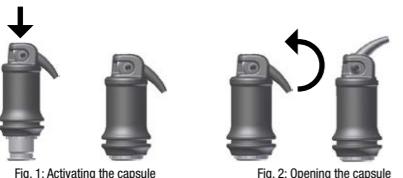
Fig. 1: Activating the capsule

Fig. 2: Opening the capsule

Luting of restorative pieces:
 Apply the material onto the prepared area and/or into the restorative piece. Then place the restorative piece in situ and let it set under slight pressure.

Any unnecessary waiting times between activation, mixing and application should be avoided, as the process of curing may impede extraction of the material and positioning of the restoration.

ORBIS GI-Cem Luting Resin has a working time of approx. 2 min at room temperature and sets in the mouth in 3 - 5 min. After approx. 4 min the material reaches a gel-like consistency, making it easy to remove any excess material with the aid of a probe. Higher temperatures shorten and lower temperatures extend these working times.

Luting of posts:

Apply ORBIS GI-Cem Luting Resin only to the post and then place the post directly into the root canal. Vibrate the post slightly during placement to avoid trapping air. Do not apply the cement into the canal with a lentulo since this could increase the speed of setting.

Usable quantity per capsule	min. 0.14 ml
Net weight per capsule	min. 0.27 g
Mixing time	10 s
Working time	approx. 2 min
Setting time	3 - 5 min

Information, precautionary measures:

Ensure that the application capsule is mixed immediately after activation.

Avoid contact of ORBIS GI-Cem Luting Resin with eugenol, thymol or other substances which inhibit the setting of ORBIS GI-Cem Luting Resin. Use temporary pieces with eugenol-free temporary cements. Before final luting, completely remove all residues of varnishes and other substances (e.g. insulating agents) from the dental hard substance as they may impair adhesion.

Ensure sufficient residual dentine thickness to prevent pulp irritation. If necessary, protect the pulp or build up the core.

Rinse instruments immediately with cold water before the cement residues have set. After removal of orthodontic bands, remove cement residues quickly and gently with an ultrasonic scaler.

Avoid contact of ORBIS GI-Cem Luting Resin with soft tissue.

ORBIS GI-Cem Luting Resin contains polyacrylic acid, peroxide and methacrylate.

Meron Plus AC should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

In case of eye contact, rinse thoroughly with much water and contact an eye specialist.

Storage instructions and application method:

Store ORBIS GI-Cem Luting Resin in a dry place at temperatures of 15 °C to 23 °C. Only remove it from the foil immediately before use. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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