ORBIS

Fluoride Varnish

Orbis Fluoride Varnish
Instructions for use

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Last revised: 06/2020

PRODUCT DESCRIPTION:

Orbis Fluoride Varnish is a fluoride-containing dental desensitisation varnish for surface application to enamel and dentine. **Orbis Fluoride Varnish** adheres to moist surfaces and is tolerant to moisture and saliva. **Orbis Fluoride Varnish** is an ethanolic suspension of colophony with artificial flavouring and xylitol sweetening. 1 ml of **Orbis Fluoride Varnish** contains 50 mg of sodium fluoride equivalent to 22.6 mg of fluoride.

INDICATIONS:

Orbis Fluoride Varnish is intended for use for:

1. treatment of hypersensitive teeth
2. the treatment of sensitive root surfaces

METHODS OF USE:

Recommended dosage:

Treatment of patients with primary dentition approx. 0.25 ml of **Orbis Fluoride Varnish**.
Treatment of patients with adolescent or permanent dentition approx. 0.40 ml of **Orbis Fluoride Varnish**.

Orbis Fluoride Varnish is applied as a thin film on the dental hard substance. Thick films peel off easily and do not provide therapeutic advantages.

By experience, the required amount for a complete set of teeth ranges between 0.25 ml and 0.40 ml.

APPLICATION:

1. Thoroughly clean tartar from treatment areas.
2. To improve results, eliminate any excessive moisture/saliva from area to be treated.
3. Uniformly apply **Orbis Fluoride Varnish** as a thin film, covering the entire target surface.
4. Moist area of application (either by gentle rinsing or natural salivary flow) to ensure proper setting of **Orbis Fluoride Varnish**.
5. Instruct patient to avoid solids, alcohol, brushing and flossing for 4 hours after application.

INFORMATION/PRECAUTIONARY MEASURES:

Orbis Fluoride Varnish contains colophony.

Orbis Fluoride Varnish should not be used in case of known hypersensitivities (allergies) to this ingredient.

Colophony may inhibit polymerisation and adhesion of composites.

According to its purpose **Orbis Fluoride Varnish** has a sticky consistency. Material remaining on the screw thread of the tube may cause the cap to stick. Wipe off any contaminating residues with an ethanol-containing cloth.

Orbis Fluoride Varnish is contraindicated in patients with ulcerative gingivitis or stomatitis. **Orbis Fluoride Varnish** is not intended for systemic treatment.

Oedematous swellings have been reported on rare occasions especially following application to extensive surfaces.

Dyspnoea in asthmatic children has been reported for comparable preparations in extremely rare cases.

Patients with sensitive digestive systems may experience nausea, usually after extensive treatment.

In case of hypersensitivity, remove varnish with a soft toothbrush and floss, then rinse with warm water.

No application of other fluoride preparations should be carried out on day of treatment. Suspend treatment with fluoride tablets for several days after treatment with **Orbis Fluoride Varnish**.

Do not swallow the varnish.

STORAGE:

Store at temperatures between 4°C - 23°C. Do not use **Orbis Fluoride Varnish** after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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MP Medical Product GmbH
Distributed by Plandent Oy