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Instructions for Use

Intended Purpose

The ergonomical designed ORBIS Cap Applicator is for intraoral application of two component dental materials (e.g. ORBIS MTA cement or glass ionomer cements) in ORBIS Caps. It is a reusable product.

Indications

Application of materials in ORBIS Caps (e.g. ORBIS MTA cement glass ionomer cements)

Contraindications and side effects

None known

General Information

This applicator is delivered non-sterile. Keep the ORBIS Cap Applicator in a dry place to prevent from corrosion.

Use of the ORBIS Cap Applicator (see Instruction for activating and mixing ORBIS Caps)

Warnings/Safety Information

- Avoid fast extrusion of the material from the ORBIS Cap.
- Do not use excessive force when moving the lever.
- Before each use check the functionality (ease of movement)

Direction for processing according EN ISO 17664

Reprocessing

Disassembly of the ORBIS Cap Applicator is not necessary.

Warnings

Please follow the country-specific regulations on hygiene standards and cleaning/disinfection of medical devices in dental practices.

Limitations on processing

- Any ORBIS Cap Applicator showing visible damage or wear must not longer be used.
- End of life of a reusable dispenser is normally determined by wear and damage due to use.
- Use-life of the dispenser is exceeded if visual inspection shows unacceptable deterioration like wear, discoloration, crackles, etc.

Initial treatment at the point of use

- Remaining cement should be removed immediately.
- Immediately after use wipe the ORBIS Cap Applicator with a lint-free cellulose tissue.
- The dispenser should be transported to the area for cleaning and sterilization in a suitable transport box or placed in a solution from detergent and disinfection.

Preparation

Wear suitable gloves, water repellent protective skirt and protective glasses for cleaning and disinfection.

Note: It is recommended to use a disinfectant solution which contains rust inhibitor.

Manual cleaning

- Prepare the cleaner according to manufacturer instructions.
- Immerse the ORBIS Cap Applicator completely in the cleaning solution and carry out all further steps below the liquid level to prevent splashing of the contaminated liquid
- Clean critical parts of the ORBIS Cap Applicator for 1 Minute with a soft brush.
- Move all moveable parts several times in the cleaning solution.
- Leave the ORBIS Cap Applicator in the cleaning solution for at least 5 minutes.
- For completely removal of the cleaning solution rinse the dispenser under running tap water (drinking water quality). Move all moveable parts several times.

Check the ORBIS Cap Applicator on cleanliness. If necessary, repeat all above steps.

Manual disinfection

Disinfection with alcohol or 1,5 % glutaraldehyd-containing mixtures for disinfection or sterilization is possible (max. 1 hour).

- Prepare the disinfectant to manufacturer instructions.
- Immerse the ORBIS Cap Applicator completely in the disinfectant and carry out all further steps below the liquid level to prevent splashing of the contaminated liquid.
- Move all moveable parts several times in the cleaning solution.
- For exposure time refer to manufacturer instructions.
- For completely removal of the disinfectant, rinse the dispenser under running tap water (drinking water quality). Move all moveable parts several times.
- Wipe with a lint free tissue or dry with clean compressed air.

Drying

- Allow the ORBIS Cap Applicator to dry until visibly dry.

Maintenance, Inspection and Testing

Inspect the ORBIS Cap Applicator under normal lighting:

- Any ORBIS Cap Applicator showing visible damage or wear must not longer be used.
- Repeat manual cleaning and inspection if the ORBIS Cap Applicator is not visibly clean.

Sterilization

The ORBIS Cap Applicator can be sterilized by hot steam (134°C/273 °F for 3 – 3.5 minutes).

- Place the ORBIS Cap Applicator in the sterilizer chamber.
- Start the sterilization program.
- At the end of the sterilization program remove the ORBIS Cap Applicator and let it cool down.

Warranty of the ORBIS Cap Applicator will be terminated after 2 years from date of delivery.

For dental use only!

Distributed by:


ORBIS DENTAL Handelsgesellschaft mbH
Schuckertstr. 21 | D-48153 Münster | Germany
Tel. 0251/322678-0 | www.orbis-dental.de

Manufacturer:
First Scientific Dental GmbH
25335 Elmshorn, Germany


Instruction for activating and mixing ORBIS Caps

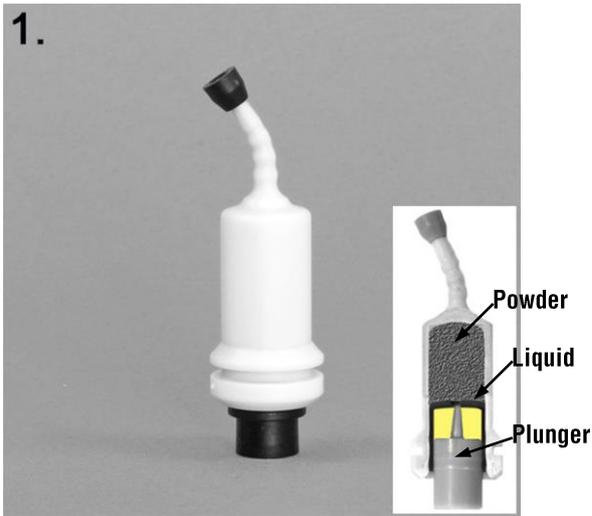ORBIS Cap before activation.


For activation of the ORBIS Cap press the plunger on a hard and plane surface to the end into the ORBIS Cap.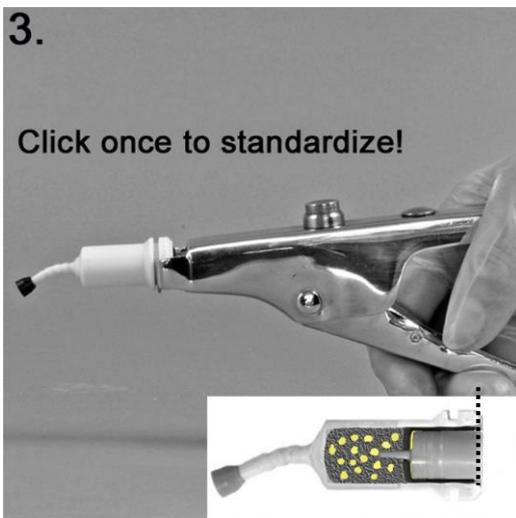


Insert the ORBIS Cap into the ORBIS Cap Applicator and **click once** to standardize.


Note: The plunger must be at the same level as the bottom of the capsule.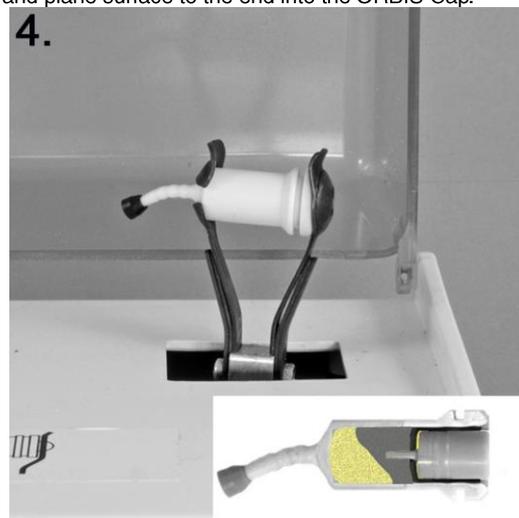

Insert the ORBIS Cap into a mixer (or an amalgamator), close lid and mix immediately (about 4300 oscillations / min). Mixing time: see corresponding instructions for use.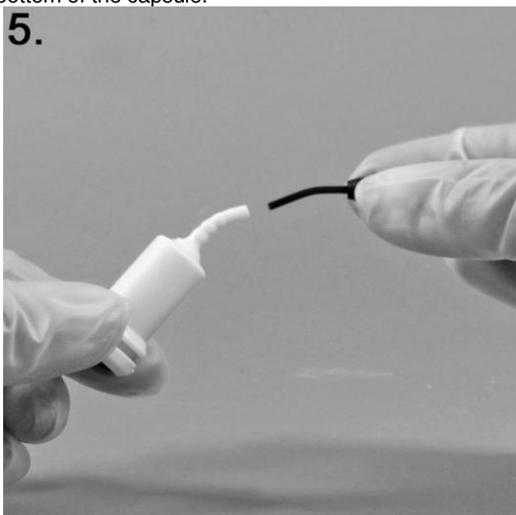

Remove the pin from the nozzle. If not, capsule can burst.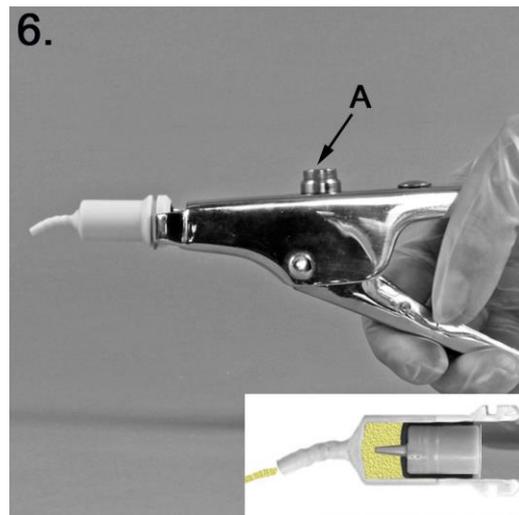

Insert the ORBIS Cap into the ORBIS Cap Applicator. Pull the lever 2 times (2 clicks) to prime the ORBIS Cap. Extrude the mixed material directly into the preparation. Unlock the gun (push button A) and remove the ORBIS Cap.

Only with the ORBIS Cap Applicator the optimal amount of mixed material is guaranteed.