EN DE SV DA NO FI

ORBIS GI-Cem Luting

EN INSTRUCTIONS FOR USE
MD EU Medical device
In accordance with DIN EN ISO 9917-1

Product description:

ORBIS GI-Cem Luting is a radiopaque glass ionomer cement for luting in application capsules for direct application. ORBIS GI-Cem Luting is characterised by good rheological properties, a high level of compressive strength, fluoride release and a low film-thickness.

Each capsule contains a minimum of 0.19 ml of usable cement. Shades: universal white.

Indications:

- Luting of metal-based crowns, bridges, inlays and onlays
- Luting of high-strength all-ceramic crowns and bridges (such as aluminium oxide, lithium disilicate and zirconium dioxide ceramics)
- Luting of metal-based and high-strength all-ceramic crowns and bridges (such as aluminium oxide, lithium disilicate and zirconium dioxide ceramics) on implant abutments
- Luting of metal root posts or indirect metal post core build-ups
- Luting of prefabricated steel crowns
- Luting of orthodontic bands

Method of Use:

Preparation:

Pre-treatment of the restoration according to the manufacturer's instructions (e.g. sandblasting of the inside of metal restorations). If necessary, degrease using alcohol or acetone. Cover the dentine near to the pulp with a suitable calcium hydroxide preparation. Clean the prepared cores (e.g. with a cleaning paste), rinse with water and dry. Do not overdry the dentine. The surface should appear slightly moist.

Activating the application capsule and mixing the material:

Only extract the application capsule from the tubular bag immediately before use. Shake the application capsule briefly by hand before use, in order to loosen the powder. During activation, the liquid from the plunger must be pressed into the inside of the capsule as fully as possible. To achieve this, the end of the application capsule where the plunger of liquid protrudes is pressed down onto a hard surface (e.g. a table) as far as the plunger will move (see fig. 1). Only activate the application capsule immediately before use. Load the activated application capsule immediately into a high-frequency mixer with approx. 4,000 oscillations per minute and allow it to mix for 10 seconds. Remove the application capsule from the mixer immediately and apply the material. Working time and consistency may be influenced by the respective type of capsule mixer.

Fig. 1: Activating the application capsule

Application:

Carefully open the outlet by lifting the application tip until you feel a stop (see fig. 2). Place the opened application capsule into the holding bracket of an application instrument (see respective instructions for use). The cannula can be aligned as required by turning the whole application capsule in the holder. By operating the lever, dispense the material directly onto the surfaces which have been prepared for luting and distribute it using, for example, the application tip of the application capsule. Always apply a slight excess of material. Insert the restoration immediately applying light pressure and hold it in position continuing to apply light pressure while curing takes place. Avoid unnecessary waiting times between activation, mixing and application, since the inefficient setting may make dispensing and the positioning of the restoration more difficult.

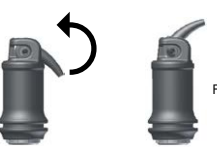
Fig. 2: Opening the application capsule

At room temperature, ORBIS GI-Cem Luting allows a window of approx. 3 minutes working time, before setting begins after 5 to 7 minutes (3 - 5 mins. at 37 °C). Higher temperatures shorten and lower temperatures extend these working times. Prevent the ingress of water and saliva during application and setting phase. Excess material can be removed particularly easily during the rubber-elastic phase (before complete curing) using a scaler or a curette, for example.

Information, precautionary measures:

ORBIS GI-Cem Luting contains polyacrylic acid. ORBIS GI-Cem Luting should not be used in case of known hypersensitivities (allergies) to this ingredient. Avoid direct application to dentine near the pulp or to exposed pulp. In case of contact with the eyes, rinse immediately with plenty of water and consult a medical specialist. ORBIS GI-Cem Luting is not intended for non-reinforced all-ceramic restorations such as silicate ceramics, nor for indirect composite restorations, orthodontic brackets, ceramic inlays, ceramic onlays or veneers. Instruments used need to be rinsed immediately with cold water before residues of material cure. Following the removal of orthodontic bands, residues of material can be removed quickly and gently using an ultrasonic scaler. Ensure that the application capsule is mixed immediately after activation.

Storage instructions and application method:
Store ORBIS GI-Cem Luting in a dry place at temperatures of 15 °C to 23 °C. Only remove it from the foil immediately before use. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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