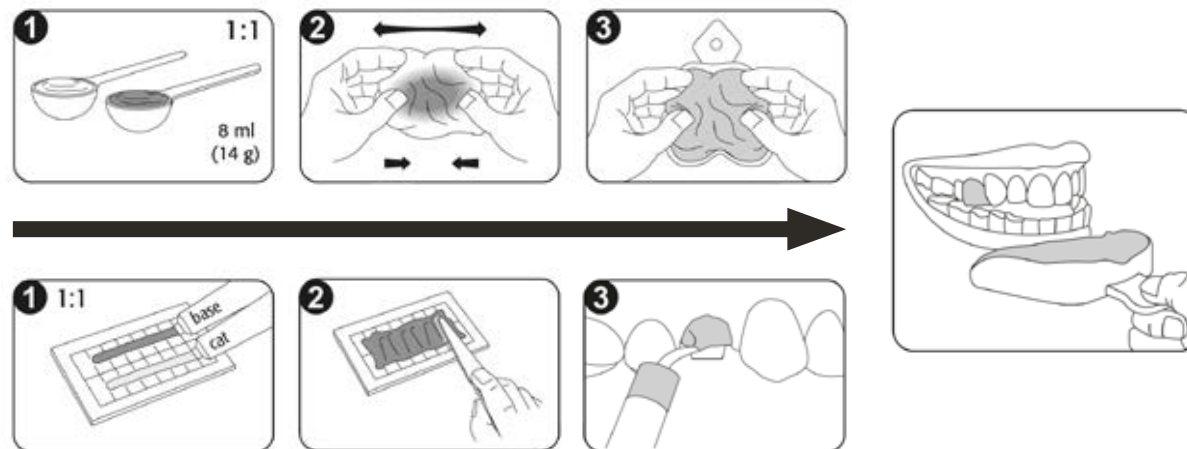
ORBIS))) A-Silikon classic

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en Instructions for use

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de

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en

Technical specifications	A-Silikon classic	Correction material	Correction material
Consistency / EN ISO 4823	Kneadable Type 0	Light-bodied Type 3	Medium-bodied Type 2
Colour	Light Green / Blue	Pink	Lilac
mixing time	30 sec.	Automatic	Automatic
Working time incl. mixing time*	Up to 2 min.	Up to 2 min.	Up to 2 min.
Time in mouth	≥ 2 min.	≥ 2 min.	≥ 2 min.
Total setting time	4 min.	4 min.	4 min.
Deformation under pressure (approx.)	2.0 %	3.0 %	2.0 %
Recovery from deformation (approx.)	99.4 %	99.6 %	99.6 %
Linear dimensional change (approx.)	0.20 %	0.16 %	0.13 %
Flavour	Mint	Mint	Mint

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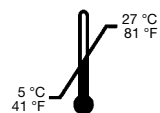
en At 23°C / 73.4°F. The times specified in the table above are based on a relative humidity of 50 +/- 10%. As a general rule: higher temperatures accelerate setting and lower temperatures delay it.

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A-Silikon classic

Elastomeric precision impression material on vinyl polysiloxane basis, addition-curing

Thank you for the confidence you have shown in our company. You have made the right decision with the ORBIS A-Silikon impression system. Please always read these instructions for use through carefully before using the product! ORBIS A-Silikon products are only intended for use in dental applications by correspondingly qualified professionals. The medical device must be employed responsibly and with due caution

Product description: ORBIS A-Silikon is an impression material which satisfies the highest demands. The putty-wash materials offer very high thixotropism, outstanding resilience and hydrophilic properties. All ORBIS A-Silikon materials have a thermo-activating formulation which offers the user a variable processing time and thus the greatest processing comfort. The time in mouth is patient-friendly as it is always equally short. The system is rounded off by the highly contrasting colour system and the pleasant mild mint flavour.

Indication: Crowns and bridges, partial crowns, inlays, onlays, implants, transfer impressions, functional impressions.

Impression technique:

- A-Silicone classic Preliminary impression material for the putty-wash technique (2-step), situation impression.
- A-Silicone correction material: wash material for putty-wash technique and double mixing technique.

Impression procedure: When using a suitable tray adhesive, please follow the manufacturer's instructions for use. After inserting the tray with the ORBIS A-Silikon materials into the patient's mouth, press it into place briefly and then hold in place until fully cured. Always check the material intraorally to assess the degree of curing. When using the putty-wash technique (2-step), the first impression must be cleaned and dried in order to ensure cross-linking with the wash material.

Disinfection: The impressions can be disinfected with a 2 % glutaraldehyde solution. After removing the impression from the mouth, rinse it under running water for 15 seconds. Then disinfect the impression by immersing it in the disinfectant thoroughly. Observing the disin-

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fectant manufacturer's specifications is essential. We recommend the aldehyde-free product ORBI-Sept Impression disinfectant.

Electroplating: The impressions can be electroplated with copper or silver in standard baths.

Model fabrication: The impression can be poured after just 60 mins. There are no other time restrictions. All class 3 and 4 gypsums which comply with the requirements in ISO 6873, e.g., ORBIS Occlumedanit, ORBI-Medanit S, ORBIS Occluroc (Type 3) and ORBIS Occlu-Superstone, Occludur, Occlu-Sockelgips, ORBI-Scan Stone, Occlustone Premium (Type 4), can be used. Undamaged impressions can be poured several times.

Working and safety instructions: The products described above must only be used in accordance with their instructions for use. The user has sole liability for any other use which contravenes these instructions for use.

- Close containers and tubes tightly immediately after use and do not interchange the lids and measuring spoons.
- Products marked “single use” are intended for a single use. To avoid cross-contamination, disinfect cartridges and mixing guns before the next use, e.g., with ORBI-Sept Wet Wipes.
- Solutions used prior to impression taking, such as mouth rinses or retraction solutions, may interfere with the setting reaction and must be removed thoroughly.
- Contact between the material and certain types of gloves, e.g., latex gloves, should be avoided as the catalyst can be damaged. We recommend nitril / vinyl or PE-based gloves.
- Do not leave any impression material residues in the oral cavity.
- Avoid contact with clothing, as cross-linked silicones are chemically resistant and leave stains which cannot be removed.
- Do not use ORBIS A-Silikon products with polyether, polysulphides or condensation-curing silicones.
- To prevent possible incompatibilities, do not combine ORBIS A-Silikon products with A-silicones from other manufacturers.
- Storage: Store in a dry place at between 5°C and 27°C (41°F and 80.6 °F).
- Shelf life: See date printed on package / label. Do not use after expiry date.
- Processing temperature: Room temperature between 18°C and 25°C (64.4°F to 77°F).

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Contraindication: The product must not be used in patients with known allergies to flavourings or other ingredients (see safety data sheet or ask the manufacturer). This should be clarified before use!

Warnings:

- Eye contact:** Avoid contact with eyes. In case of contact, rinse eyes immediately with plenty of clean water and consult an ophthalmologist without delay.
- Swallowing:** If swallowed, rinse out the mouth and drink plenty of water. Always contact a doctor.
- Allergic reactions:** It is not possible to rule out allergic reactions in hypersensitive people. If acute allergic reactions should occur during the treatment, treatment with the product should be stopped immediately and a doctor should be consulted.

Should a serious incident occur unexpectedly in connection with this medical device, it must be reported to the manufacturer and the competent authority in your Member State.

Please also observe the warnings in the safety data sheet!

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